




Blood Serum Preparation for Autoantibodies determination

- ✓ for all tests required it's enough to send one blood collection tube
- ✓ please use collection tubes for SERUM (see picture below)



BD Vacutainer® Venous Blood Collection Tube Guide

For the full array of BD Vacutainer® Blood Collection Tubes, visit www.bd.com/vacutainer. Many are available in a variety of sizes and draw volumes (for pediatric applications). Refer to our website

BD Vacutainer® Tubes with BD Hemogard™ Closure	BD Vacutainer® Tubes with Conventional Stopper	Additive	Inversions at Blood Collection*	Laboratory Use
 Gold	 Red/Gray	<ul style="list-style-type: none"> • Clot activator and gel for serum separation 	5	For serum determinations in chemistry. May be used for routine blood donor screening and diagnostic testing of serum for infectious disease.** Tube inversions ensure mixing of clot activator with blood. Blood clotting time: 30 minutes.

- ✓ mixing recommendation: tubes should be gently inverted 180° and back 5-6 times
- ✓ the minimum recommended clotting time for patients without anticoagulant is 30 minutes
- ✓ centrifugation: 1300-2000 g (rpm depends on your centrifuge) for 10 minutes at 25°C room temperature
- ✓ filling order form (together with treating doctor)

Shipping:

- ✓ to ensure sample(s) will arrive our lab before weekend, please send sample(s) at the beginning of the week
- ✓ sample(s) don't have to be cooled
(exception: at temperatures over 25°C/77°F (in Germany usually from May till August) please use a box with a household common freezed coolpad)
- ✓ use a courier service like FedEx (International First or International Priority)
- ✓ send sample(s) to the following address:
CellTrend GmbH
Im Biotechnologiepark 3 (TGZ 2)
14943 Luckenwalde
GERMANY
- ✓ please send an email with tracking info: lab-service@celltrend.de

Results & Billing:

- ✓ invoice will be send per email within two weeks after sample(s) sending
- ✓ test results will be send after balancing the invoice (please write email-address in CAPITAL LETTERS)
- ✓ payment is possible by PayPal or wire transfer to our bank account